Deutsch

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Date of issue: 15 June 2016

Intended use:

Therapon Transpa are highly transparent PMMA blanks conceived for milling operations in dental technology. These blanks have been realised for the manufacturing of bite splints, orthodontic splints and occlusal splints for bruxism, and are suitable for long-term use. This product can be milled with any Zirkonzahn milling unit using the appropriate milling tools and parameters.

Contraindications:

Do not use the product in conjunction with cloves, nutmeg, pimento and teas containing essential oils.

Instructions of use:

- 1) Fix the resin blank into the clamping device provided and start the milling process.
- 2) After milling, check the structure visually for material fractures or cracks. Only perfectly milled frameworks can be used as bite splints, orthodontic splints and occlusal splints for bruxism.
- 3) Proceed with the mechanical processing: finish the structure using a wide cross-cut tungsten-carbide bur first, and then a fine one, at 15,000 rpm.
Polishing: use polishing brushes at 5,000 rpm, then buff with pumice at 1,500 rpm and finally high-gloss polish with goat hair brushes and polishing paste at 5,000 rpm.
Important: do not overheat the resin when using burs, brushes, steam jets etc.
- 4) Clean the final job from dust (steam jets) and sterilize it.

Advices:

The resin blanks are already polymerised when supplied, but avoid dropping them. Product quality and durability can be compromised if handled improperly. Dust from grinding can irritate eyes, skin or mucous membranes. Take protective measures. Show intolerance to the product be suspected, use it only after an allergy screening test has been performed with a medical statement issue of non-allergy conditions. Disinfect the milled parts before inserting them into the patient's mouth. Implant hygiene is the patient's responsibility. These instructions for use correspond to the current state of technology and our own experiences. The user will be held fully responsible for the correct use of the product. The manufacturer has no influence on the processing procedures, therefore he is not responsible for any unsatisfactory results. The user (dental technician) shall inform the dentist as regards the operations to be performed before inserting the part into the patient's mouth, making sure that the dentist has received all of the information contained in these instructions for use. Therefore, any claims for damages shall concern exclusively the commercial value of our products.

Technical Data:

Strength:
Elasticity modulus 3.300 MPa
Flexural strength 115 MPa
Ultimate tensile strength 80 MPa
Deformation at break 5.5 %

Storage and stability:

Store in a cool, dry place, do not expose to physical impact or vibrations, avoid contact with liquids, protect the blanks from external agents.

Information:

Producer: Zirkonzahn S.r.l.
An der Ahr 7 – 39030 Gais – Italy
www.zirkonzahn.com
Supplier: Zirkonzahn S.r.l.
An der Ahr 7 – 39030 Gais – Italy
www.zirkonzahn.com
Emergency phone: +39 0474 066 660
(Mon. – Fri. 8 am – 6 pm)